

## EIA PCP IgG



**EAN Code:** 8595635303869

**Catalog number:** PCG096

**Package size:** 96 tests

**Storage:** 2-8 °C

**Producer:** TestLine Clinical Diagnostics s.r.o.

### Description:

- PCP antigen (1-5, 6B, 7F, 8, 9N, 9V, 10A, 11A, 12F, 14, 15B, 17F, 18C, 19F, 19A, 20, 22F, 23F, 33F).
- If specific antibodies are present, they bind to the antigen, are labeled by the Conjugate in the following steps and are detected by color reaction with a single component substrate (TMB-Complete).
- The kit allows 96 tests, including controls in a split microtiter plate with color-coded strips and breakable wells.

### Advantages:

- The total assay time is about 2 hours.
- High sensitivity and specificity of the test.
- Kit includes CUT-OFF, Positive Control, Negative Control and Calibrators (2, 6, 80, 270 U/ml).
- Semi-quantitative evaluation in the Index of Positivity (IP) or quantitative evaluation in U/ml.
- Ready-to-use, color-coded components.
- Single-component substrate.
- Interchangeable components with the exception of kit specific components (Controls, Conjugate,

Plate).

**Application:**

- Diagnostic of post-vaccination antibody response.
- Serological determination of antibodies against pneumococcal infections.

**Brief assay procedure:**

1. Dilute samples (1:101).
2. Pipette Controls and diluted Samples.
3. Incubate at 37°C for 30 min.
4. Aspirate and wash the wells 5×
5. Pipette Conjugate.
6. Incubate at 37°C for 30 min.
7. Aspirate and wash the wells 5×
8. Pipette Substrate (TMB-Complete).
9. Incubate at 37°C for 30 min.
10. Pipette Stop Solution.
11. Read color intensity at 450 nm.
12. Evaluate the test.