

## INDOXYLstrip



**EAN Code:** 8595635303395

**Catalog number:** Is0100

**Package size:** 100 tests

**Storage:** 2-8 °C

**Producer:** TestLine Clinical Diagnostics s.r.o.

### Description:

- INDOXYLstrip is a test for the detection of the acetate esterase activity of microorganisms. The test is based on the reaction of free indoxyl groups with oxygen, which results in a colour change. Acetate esterase activity is present e.g. in *Campylobacter* spp., and in *Branhamella catarrhalis*. The test is suitable for screening examinations and for the suspect microbial colonies identification.
- Shelf life: 24 months.

### Application:

- Microbiological diagnostic test for rapid identification of acetate esterase activity of *Moraxella* (*Branhamella*) *catarrhalis*, and other microorganisms.

### Brief assay procedure:

1. After the 18-24 h growth, wipe several bacterial colonies from a Petri dish using the active paper zone of the diagnostic strip.
2. Read the result after 3- 5 min.
3. Evaluate results.