

# **SmartEIA Gliadin DA IgA**



**EAN Code:** 8595635306549 **Catalog number:** SK-GDA096

Package size: 96 tests

Storage: 2-8 °C

Producer: TestLine Clinical Diagnostics s.r.o.

## **Description:**

- Microtitre wells are coated with the deamidated gliadin.
- If specific antibodies are present, they bind to the antigen, are labeled by the Conjugate in the following steps and are detected by color reaction with a single component substrate (TMB-Complete).
- The kit allows 96 tests, including controls in a split microtiter plate with color-coded strips and breakable wells.

## **Advantages:**

- The total assay time is about 1 hour 30 min.
- High sensitivity and specificity of the test.
- Kit includes CUT-OFF, Positive Control and Negative Control, Calibrators.
- Semi-quantitative evaluation in the Index of Positivity (IP).
- Ready-to-use, color-coded components.
- Single-component substrate.
- Interchangeable components with the exception of kit specific components (Controls, Conjugate, Plate).

Exportováno 1. 12. 2025 Stránka 1 / 2



## **Application:**

- Screening test for the detection of the coeliac disease.
- Monitoring the effect of the gluten-free diet.

#### **Brief assay procedure:**

- 1. Dilute samples (1:101).
- 2. Pipette Controls and diluted Samples.
- 3. Incubate at 37°C for 30 min.
- 4. Aspirate and wash the wells 5×
- 5. Pipette Conjugate.
- 6. Incubate at 37°C for 30 min.
- 7. Aspirate and wash the wells 5×
- 8. Pipette Substrate (TMB-Complete).
- 9. Incubate at 37°C for 15 min.
- 10. Pipette Stop Solution.
- 11. Read color intensity at 450 nm.
- 12. Evaluate the test.

SmartEIA kits are specifically designed for automated processing on the Agility® instrument, Dynex Technologies, Inc.

Exportováno 1. 12. 2025 Stránka 2 / 2