

# **SmartEIA Parvovirus B19 IgG**

**EAN Code:** 8595635310539 **Catalog number:** SK-PVG096

Package size: 96 tests

Storage: 2-8 °C

**Producer:** TestLine Clinical Diagnostics s.r.o.

## Description:

- Antigen used is VP2 recombinant protein.
- If specific antibodies are present, they bind to the antigen, are labeled by the Conjugate in the following steps and are detected by color reaction with a single component substrate (TMB-Complete).
- The kit allows 96 tests, including controls in a split microtiter plate with color-coded strips and breakable wells.

#### **Advantages:**

- The total assay time is about 2 hours.
- High sensitivity and specificity of the test.
- Kit includes CUT-OFF, Positive Control, Negative Control and Calibrators (1, 5, 20, 80 IU/ml).
- Semi-quantitative evaluation in the Index of Positivity (IP) or quantitative evaluation in IU/ml.
- Ready-to-use, color-coded components.
- Single-component substrate.
- Interchangeable components with the exception of kit specific components (Controls, Conjugate, Plate).

### **Application:**

- Determination of specific IgG antibodies in human serum or plasma.
- Semiquantitative evaluation is suitable for the therapy success follow-up.
- Disease stage evaluation.

#### **Brief assay procedure:**

- 1. Dilute samples (1:101).
- 2. Pipette Controls and diluted Samples.

Exportováno 28. 11. 2025 Stránka 1 / 2



.

- 3. Incubate at 37°C for 30 min.
- 4. Aspirate and wash the wells 5×
- 5. Pipette Conjugate.
- 6. Incubate at 37°C for 30 min.
- 7. Aspirate and wash the wells 5×
- 8. Pipette Substrate (TMB-Complete).
- 9. Incubate at 37°C for 30 min.
- 10. Pipette Stop Solution.
- 11. Read color intensity at 450 nm.
- 12. Evaluate the test.

SmartEIA kits are specifically designed for automated processing on the Agility® instrument, Dynex Technologies, Inc.

Exportováno 28. 11. 2025 Stránka 2 / 2