

TRIBUstrip



EAN Code: 8595635303593

Catalog number: Ts0300

Package size: 300 tests

Storage: 2-8 °C

Producer: TestLine Clinical Diagnostics s.r.o.

Description:

- The test principle is an enzyme hydrolysis of tributyrin. This reaction causes colour change of acidobasic indicator.
- The kit enables 300 pcs.
- Shelf life: 24 months.

Application:

- Microbiological diagnostic test for differentiation between Moraxella (Branhamella) and Neisseria genus.

Brief assay procedure:

1. Throw one tributyrin strip into the suspension of tested strain in 1 ml of buffered saline.
2. Incubate the tested sample at 37 °C (without CO₂).
3. Read the final results after 18-20 h.
4. Evaluate results.