


## UREASAtest bulk

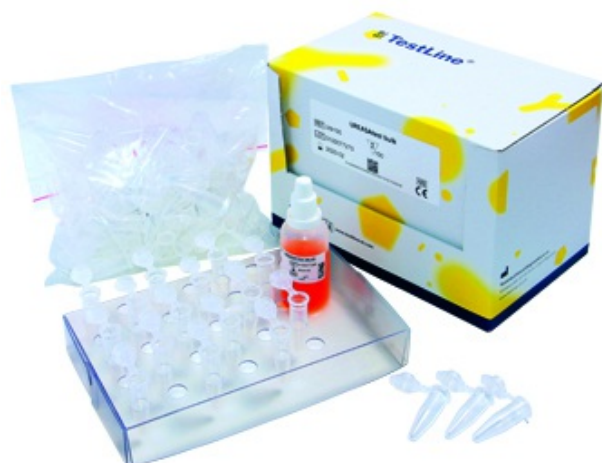

**EAN Code:** 8595635303609

**Catalog number:** Utb100

**Package size:** 100 tests

**Storage:** 2-8 °C

**Producer:** TestLine Clinical Diagnostics s.r.o.

### Description:

- UREASAtest 50 is a diagnostic medium for the rapid detection of the urease activity of *Helicobacter pylori* in biptic samples of gastric mucosa. Most strains of *H. pylori* have urease activity, i.e. they hydrolyse urea and produce ammonia. The ammonia is trapped by the medium and changes the medium's pH. The increased alkalinity of the medium causes a change in the colour of a pH indicator from yellow to red.
- Shelf life: 12 months.

### Application:

- Microbiological diagnostic medium for rapid detection of *Helicobacter pylori* urease activity in gastric mucosa biopsies.

### Brief test procedure:

1. Take necessary number of the vessels and place into the holder, mark with the number and pipette 200ul (8 drops) of the diagnostic medium into the all vessels. Close the bottle with diagnostic medium and put it back in the fridge immediately.
2. Place biptic samples of gastric mucosa into the vessels marked with sample numbers. Close the vessels. (Work under aseptic conditions.)
3. Incubate the vessels containing samples for 1 and 3 hours at room temperature.
4. Read results 1 and 3 hours after the start of the incubation.
5. Evaluate results.